

# Application Synchronization Software

## Streamlines workflow, reduces desktop complexity

Application Synchronization software, designed specifically for the SecurView® diagnostic workstation, provides patient synchronization and common login/logout with over 60 RIS, reporting, and dictation systems.

- Simplifies workflow by eliminating unnecessary typing and clicks.
- Assures that the same patient is on display in all applications, potentially reducing errors.

### Compatible with many third-party systems

The synchronization features vary by third-party systems and may:

- Automatically open and populate data to or from third-party applications on a separate display.
- Allow for SecurView workstation-driven workflow for the radiologist, opening a report or displaying associated data in the third-party application for the patient opened on the SecurView workstation.
- Allow for RIS-driven workflow for the radiologist, displaying breast images on the SecurView workstation for the patient selected on the third-party application.
- Send a message to mark studies as Read for the open patient on the SecurView workstation (for example, PowerScribe, Epic, MagView, M\*Modal Fluency, PenRad).
- Version 4.6 includes a new application type for Epic Radiant RIS, to support the encrypted XML file interface with web based Hyperdrive client applications.

Hologic's Technical Solutions Specialists will discuss compatibility with specific versions of supported third-party applications and minimum system requirements.

- Application Synchronization software is compatible with most RIS, Mammography Reporting, and Dictation Systems.
- The third-party application can be displayed on a third or fourth monitor. If a third monitor is already present as a multi-modality configuration, the user can <Alt+Tab> between applications.
- Display options include a 19" 2MP color display or a 27" LCD color display. (See MISC-01752 for hardware specifications.)
- In most cases, third-party applications can be installed directly on the SecurView diagnostic workstation, eliminating the need for multiple computer systems.
- Optionally, the third-party application can run on a separate computer, communicating with Application Synchronization via a network.
- The Application Synchronization software communicates with third-party applications using a shared file, TCP/IP, or custom interface.
- Upgrading to the most current SecurView workstation software version for optimal performance and security is recommended.